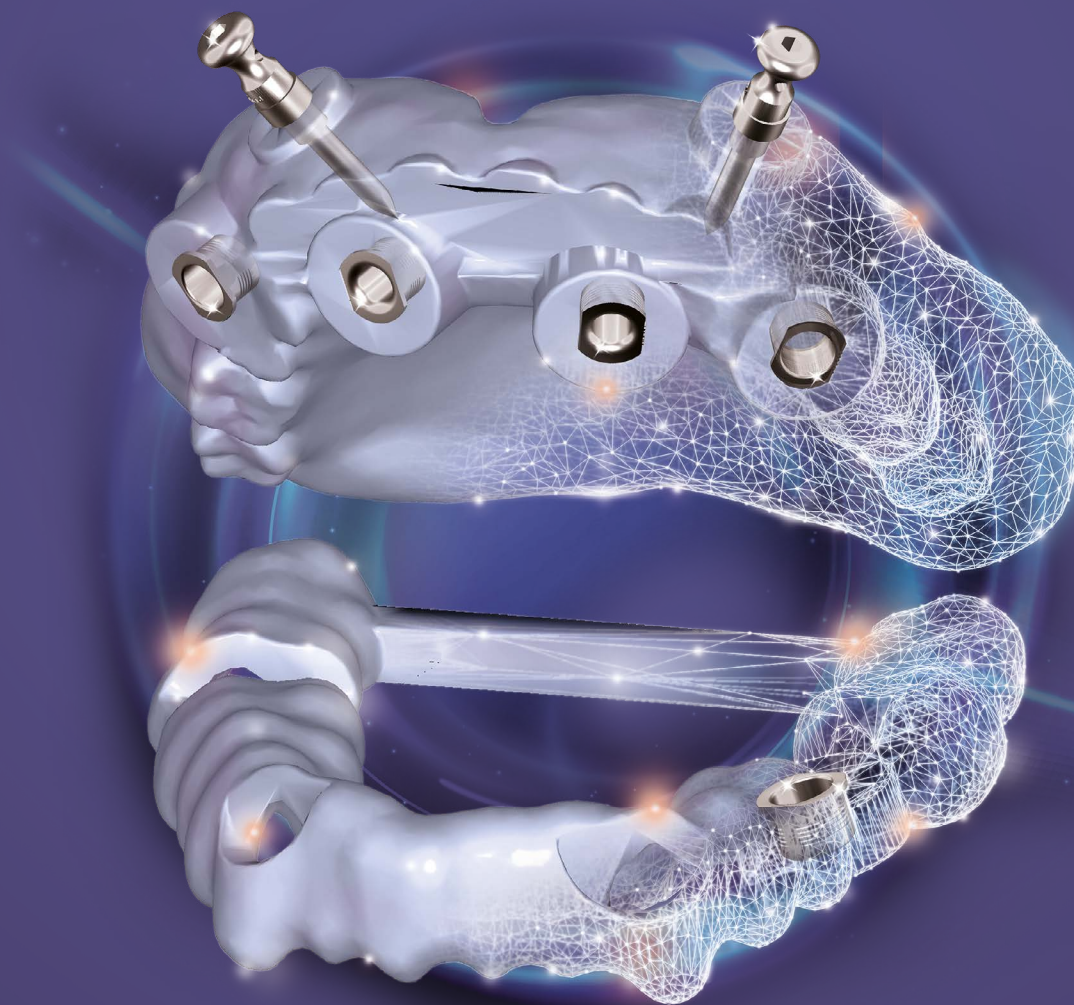
# CIRUGIA GUIADA A SU MANERA

exocad

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
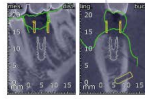
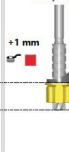
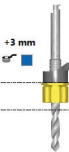
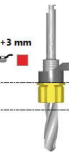
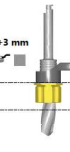
Tooth 36 (FDI)	Patient: Doe, John Project: 2020-12-01_00001-001 (Lower jaw)	This automatically generated report should only be used as a guideline and therefore must not be considered official manufacturer documentation.	exocad			
<b>BIOHORIZONS®</b> 4.6 mm Master Cylinder H6 Article Number: CSS-GMC-10 Inner Diameter Ø: 5.11 mm Height H: 5.99 mm Distance from sleeve top to platform: 10.53 mm To bone level: 10.53 mm		<b>BIOHORIZONS®</b> Mount-free Tapered Internal, Ø 4.6 mm, RBT Surface with Laser-Lok®, 12 mm Article Number: TLX4612 Platform Diameter Ø: 4.5 mm Length L: 12 mm Body Diameter: 4.6 mm	 			
<b>BIOHORIZONS® - Guided Surgery Kit CGS4000</b>						
Fully Guided						
1 Ø 4.6 mm	2 Ø 2 mm L 24 mm +1.47 mm	3 Ø 2.5 mm L 24 mm +1.47 mm	4 Ø 3.2 mm L 24 mm +1.47 mm	5 Ø 3.7 mm L 24 mm +1.47 mm	6 Ø 4.1 mm L 24 mm +1.47 mm	7 Stop Position SP3 Insert Implant up to Stop Position SP3.
						

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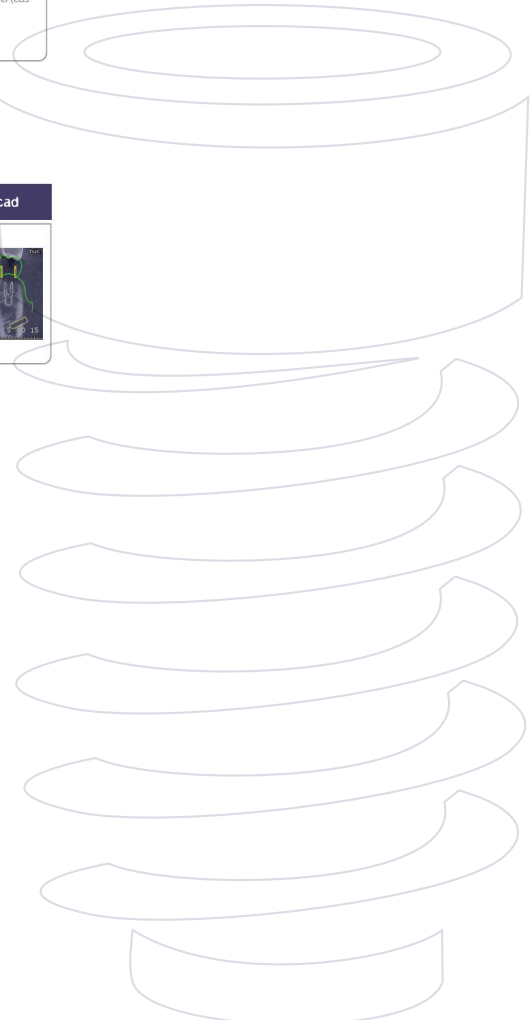
Tooth 36 (FDI)	Patient: Doe, John Project: 2020-12-01_00001-001 (Lower jaw)	This automatically generated report should only be used as a guideline and therefore must not be considered official manufacturer documentation.	exocad		
<b>straumanngroup</b> Ø 5.0 mm self locking T-sleeve Article Number: 034.299V4 Inner Diameter Ø: 5 mm Height H: 5 mm Distance from sleeve top to platform: 6.87 mm To bone level: 7 mm		<b>straumanngroup</b> BLX, Ø 4.0 mm RB, SLActive® 10 mm, Roxolid® Article Number: 061.5310 Platform Diameter Ø: 3.5 mm Length L: 10 mm Body Diameter: 4 mm	 		
<b>straumanngroup - X VeloDrill™ Guided Surgery Kit</b>					
Fully Guided Medium Bone Density					
1 Ø 4 mm Use up to Depth Mark H2.	2 Ø 3.5 mm L 8 mm +1 mm Use Milling Cutter to flatten alveolar ridge. Depth Marks every 2 mm.	3 Ø 2.2 mm L 20 mm +3 mm	4 Ø 3.5 mm L 20 mm +3 mm	5 Ø 3.7 mm L 16 mm +3 mm Coronal Bone Widening: Drill Depth 6.0 mm only	6 Depth Mark H2 Insert Implant up to H2 Depth Mark.
					

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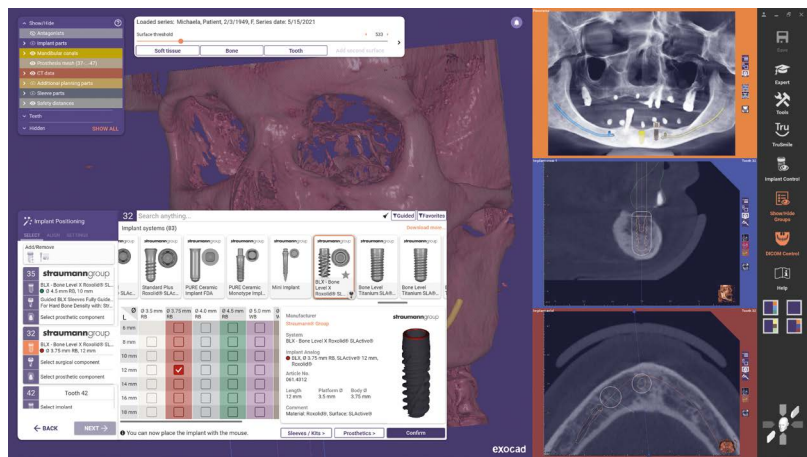**35** **straumann**group

BLX - Bone Level X Roxolid® SLActive  
● Ø 4.5 mm RB, 10 mm

Guided BLX Sleeves Fully Guide...  
For Hard Bone Density with: Str...

Select prosthetic component

**32** **straumann**group

BLX - Bone Level X Roxolid® SLActive  
● Ø 3.75 mm RB, 12 mm

Select surgical component

Select prosthetic component

**42** Tooth 42

Select implant

← BACK
NEXT →

	Ø 3.5 mm RB	Ø 3.75 mm RB	Ø 4.0 mm RB	Ø 4.5 mm RB	Ø 5.0 mm WB	Ø 6 mm
6 mm						
8 mm						
10 mm						
12 mm		<input checked="" type="checkbox"/>				
14 mm						
16 mm						
18 mm						

**straumann**group

**BLX - Bone Level X Roxolid® SLActive**

Implant Analog  
● BLX, Ø 3.75 mm RB, SLActive® 12 mm, Roxolid®

Article No.  
061.4312

Length 12 mm    Platform Ø 3.5 mm    Body Ø 3.75 mm

Comment  
Material: Roxolid®, Surface: SLActive®

Sleeves / Kits >
Prosthetics >
Confirm

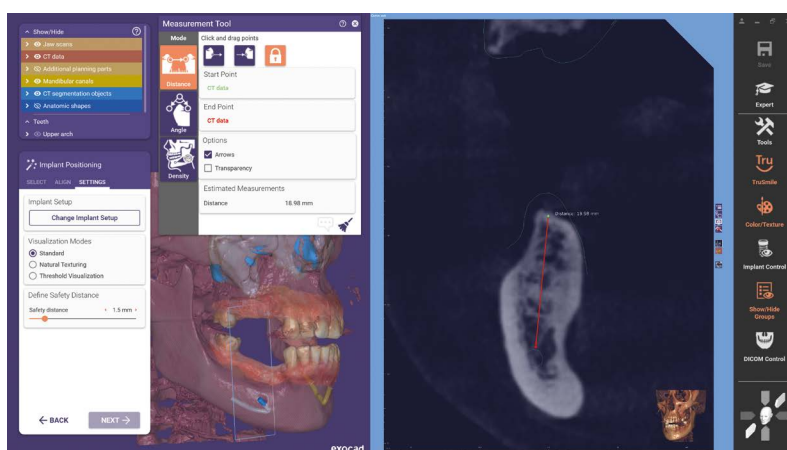
● You can now place the implant with the mouse.

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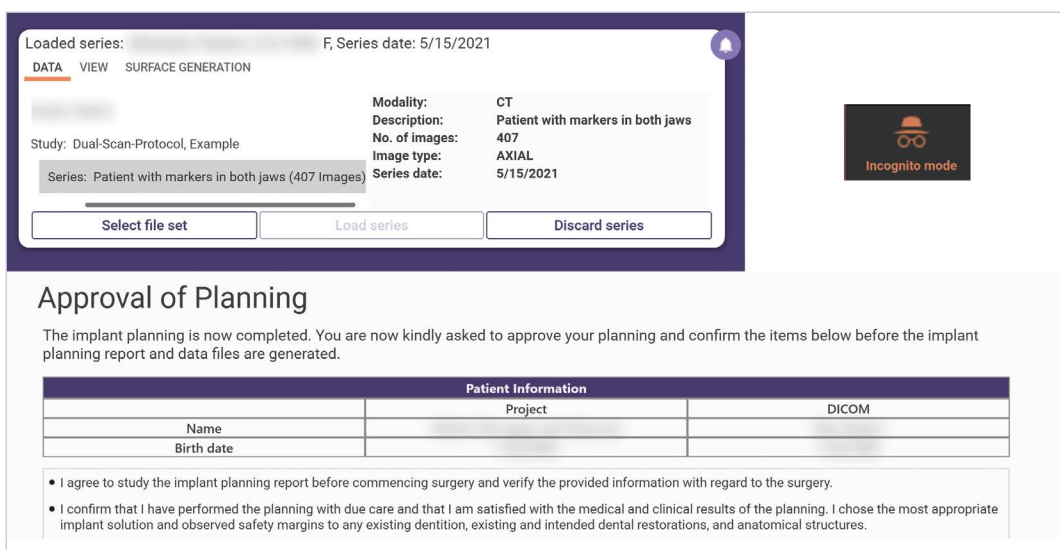
The screenshot shows a software interface with a top navigation bar containing 'DATA', 'VIEW', and 'SURFACE GENERATION'. Below this, there is a section for 'Loaded series' with the following details:

- Study: Dual-Scan-Protocol, Example
- Series: Patient with markers in both jaws (407 Images)
- Modality: CT
- Description: Patient with markers in both jaws
- No. of images: 407
- Image type: AXIAL
- Series date: 5/15/2021

Buttons for 'Select file set', 'Load series', and 'Discard series' are visible. To the right, a dark box with a white icon and the text 'Incognito mode' is displayed.

**Approval of Planning**

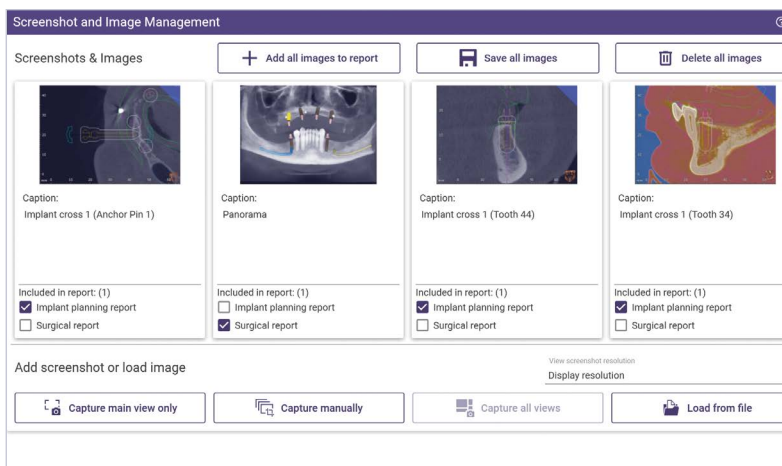
The implant planning is now completed. You are now kindly asked to approve your planning and confirm the items below before the implant planning report and data files are generated.

Patient Information		
	Project	DICOM
Name		
Birth date		

- I agree to study the implant planning report before commencing surgery and verify the provided information with regard to the surgery.
- I confirm that I have performed the planning with due care and that I am satisfied with the medical and clinical results of the planning. I chose the most appropriate implant solution and observed safety margins to any existing dentition, existing and intended dental restorations, and anatomical structures.

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The screenshot shows the 'Screenshot and Image Management' interface. It features a grid of four image thumbnails, each with a caption and report inclusion options.

Thumbnail	Caption	Included in report: (1)
	Implant cross 1 (Anchor Pin 1)	<input checked="" type="checkbox"/> Implant planning report <input type="checkbox"/> Surgical report
	Panorama	<input type="checkbox"/> Implant planning report <input checked="" type="checkbox"/> Surgical report
	Implant cross 1 (Tooth 44)	<input checked="" type="checkbox"/> Implant planning report <input type="checkbox"/> Surgical report
	Implant cross 1 (Tooth 34)	<input checked="" type="checkbox"/> Implant planning report <input type="checkbox"/> Surgical report

Buttons for 'Add screenshot or load image', 'Capture main view only', 'Capture manually', 'Capture all views', and 'Load from file' are visible at the bottom.

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